LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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GUARDIAN

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REF MDS89664 REF MDS89664A

DO NOT install this product without first reading and understanding this instruction manual. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

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GENERAL WARNINGS

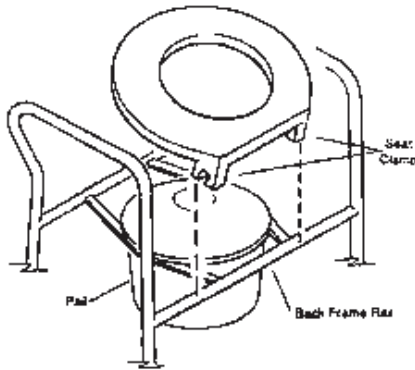
DO NOT install this product without first reading and understanding this instruction manual.

If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment.

- Toilet seat must be in down position before use. Failure to do so may result in breakage of unit and possible injury to user.
- Inspect all parts for shipping damage. In case of shipping damage, do not use. Contact dealer/carrier for further instructions.
- Make certain that the snap buttons fully protrude through the same respective adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and an even height adjustment is achieved.
- Ensure all screws, nuts and/or bolts are tight at all times.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing. Replace them immediately if any of these conditions exist.
- All four leg extensions with rubber tips must touch the floor at all times.
- Users with limited physical capabilities should be supervised or assisted when using a commode.
- DO NOT exceed the maximum Weight Capacity or serious injury could result: Maximum weight capacity: MDS89664 - 350 lbs (159 kg), MDS89664A - 300 lbs (136 kg).

**NOTE: Tips on This Product Contain Dry Natural Rubber.
FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.**

Installing and removing the commode seat and lid (Figure 2)
(Figure 2)

CAUTION:

1. Before removing/installing seat and lid, allow seat/ring to reach room temperature. This will help prevent the seat clamps from breaking when being removed/installed onto commode back frame rail.

2. Align the clamps of the toilet seat onto the back frame rail. (Each clamp should sit outside the pail holders.) Push down on seat until it clamps firmly into place.
3. Repeat the same process for the lid.
4. When removing the seat or lid, remove the lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift up.

HEIGHT ADJUSTMENT

Commode's height is adjustable for use in an elevated position.

1. Depress detent button on leg piece and move up or down to the desired position.
2. Ensure that the detent button is fully engaged and leg does not move.
3. Repeat steps 1 and 2 for the remaining three legs.
4. Make sure the commode sits level and all of the legs are at the same height.

USER INSTRUCTIONS

Use only as instructed by your healthcare professional.

SITTING AND STANDING

1. Back up to the commode until you feel the back of your knees touching it.
2. Reach back for the armrests. Place both hands firmly on the armrests before sitting on the commode.
3. Slowly lower yourself onto the commode by bending your elbows and, if possible, your hips and knees.
4. Reverse steps 2 through 4 for getting up. Push on the armrests as much as comfortably possible. Make sure you have your balance before releasing your grasp from the commode armrests.

OVER THE TOILET USE

1. Lift seat of commode and remove bucket.
2. Replace with splash guard.
3. Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.

NOTE: Check to ensure that commode is level and stable before sitting.

CLEANING

Your commode can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

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