

# Medline OVERBED TABLE



MDS107015 MDS104015 MDS104950

## Assembly and Operating Instructions

*NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instruction.*

### **SAFETY SUMMARY**

The following recommendations are made for the safe assembly of the Overbed table.

#### **WARNING**

**DO NOT** assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment—otherwise, injury or damage may occur.

After assembly and **BEFORE** use, ensure that **ALL** attaching hardware is securely tightened.

### **ASSEMBLING THE OVERBED TABLE (FIGURE 1)**

1. Position the column assembly as shown in FIGURE 1.

**NOTE:** Ensure that the release rod inside the column assembly is visible in the lower (rectangular) portion of the column assembly as shown in DETAIL "A". If necessary, shake the column assembly to allow the release rod to drop into the column assembly.

2. Insert the release lever into the column assembly as shown in FIGURE 1.

**NOTE:** The release lever **MUST** be positioned with the curved side closest to the tabletop assembly and in between the release rod and column assembly.

3. Lay the overbed tabletop assembly on a flat surface with the top bracket facing UP.

4. Hold the release lever and column assembly together and install into top bracket.

5. Align holes in column assembly with holes in top bracket.

6. Install one (1) bolt through top bracket, column assembly and release lever.

7. Install remaining bolt through top bracket and column assembly.

8. Install locknuts onto bolts in STEP 6, 7. Securely tighten.

9. Place the base assembly upside down on a flat surface with the mounting holes facing UP.

10. Install one (1) caster into the base assembly until it snaps into place.

11. Repeat STEP 10 for remaining three (3) casters.

12. Install one (1) bolt through the bolt plate, base and column assembly. Hands tighten. Repeat for other side.

13. Position overbed table on its side.

**NOTE:** Positioning the overbed table on its side will elevate pressure of the base on the bolts.


14. Securely tighten the two (2) bolts in STEP 12.

15. Position overbed table on all four (4) casters.

16. Remove the shipping screw from the column assembly and install the provided Phillips screw.

17. When adjusting tightness on the retaining bolt for the Tilt Top Overbed Table (MDS104950), over or under tightening may cause the tilt feature to not function properly.

18. Adjust the height to desired position. Refer to **ADJUSTING THE OVERBED TABLE** in this instruction sheet.

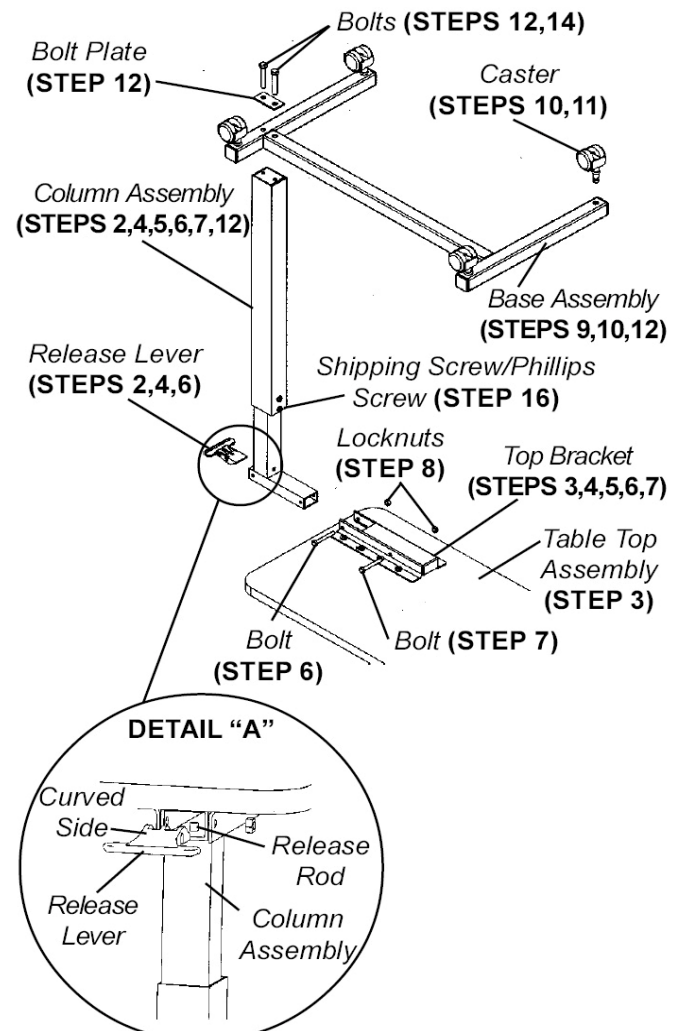

**FIGURE I - ASSEMBLING THE OVERBED TABLE**

[www.Medline.com](http://www.Medline.com)

© 2007 Medline is a registered trademark of Medline Industries, Inc.  
Manufactured for Medline Industries, Inc., Mundelein, IL 60060 USA  
Made in China 1-800-MEDLINE

## ADJUSTING THE OVERBED TABLE (FIGURE 2)

1. Do one (1) of the following
  - A. **TO RAISE** – Pull UP from the bottom of the tabletop assembly UP to the desired height.
  - B. **TO LOWER** – Pull up on the release lever and adjust the table top DOWN to the desired height.

NOTE: The column will lock into place when the release lever is released.

NOTE: When adjusting the tilt top overbed table, release the plunger knob and make sure it is fully seated prior to adjusting the height of the overbed table top. Failure to do this can result in injury to your finger or hand.

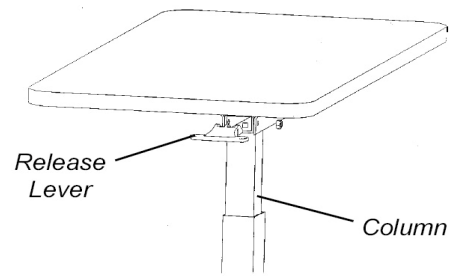


FIGURE2-ADJUSTINGTHEOVERBEDTABLE

### Medline Limited Warranty

Your Medline Product is warranted to be free of material defects in materials and workmanship for a period of **three (3) years** from the date of purchase. Non-durable components such as handgrips, pushbuttons, and leg riggings are warranted for a period of **one (1) year** from the date of purchase. This express warranty extends only to the original purchaser only. In the event of a defect or failure covered by this warranty, contact us at 1-800-MEDLINE. Medline will, at its option, repair or replace your Medline product. This warranty does not extend to damage or failure occurring as a result of owner misuse or negligence, normal wear and tear, or other causes unrelated to the manufacture and design of the product. This express warranty is Medline's sole and exclusive warranty and your sole remedy in the event of default or failure. Medline is not responsible for any consequential or incidental damages or costs arising from use of the warranted products, or any defect therein or failure thereof. Medline disclaims all other warranties including but not limited to the implied warranties of merchantability and fitness for a particular purpose.

### Also available for order from Medline:

#### 3 Types of Beds:

- **Manual (MDR107000P)** – no motors
- **Semi Electric (MDR107002L)** – manual crank for height, 1 motor for head angle, 1 motor for leg/foot angle
- **Full Electric (MDR107003L)** – 1 motor for height, 1 motor for head angle, 1 motor for leg/foot angle



#### 2 Types of Mattresses:

- **Innerspring (MDS237827) Meets CFR1633 Federal Flammability code** – coil springs inside, with layer of foam on both sides of mattress. Most popular type of mattress.
- **Foam (MDR230981)** – foam throughout. Least popular type of mattress.



#### 3 Types of Bedrails:

- **Half Rails (MDS89697)** – half the length of the bed
- **Standard Full Rails (MDS89690)** – full length of the bed; 3 bars
- **Safety Full Rails (MDS89694)** – full length of the bed; 4 bars with narrow gap for safety



[www.Medline.com](http://www.Medline.com)

© 2007 Medline is a registered trademark of Medline Industries, Inc.  
Manufactured for Medline Industries, Inc., Mundelein, IL 60060 USA  
Made in China 1-800-MEDLINE