## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada  
1-800-396-6996  
www.medline.ca | Canada@medline.com

Medline Mexico  
01-800-831-0898  
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline Industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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---

# IMPORTANT SAFEGUARDS

## READ ALL INSTRUCTIONS BEFORE USING.

Please follow these instructions carefully whether commode is used bedside or over the toilet. Commode's height is adjustable for use in an elevated position. Back of commode is removable for use over the toilet. Cantilevered arm drops for left-to-right leg movement.

### WARNING:

- Please read carefully before use. Failure to comply may result in severe injury.
- Ensure all detent buttons are properly engaged in position prior to use.
- Do not exceed the maximum weight capacity of this commode.

**250 lbs. (113 kg.)**

**Failure to follow the above instructions may result in SERIOUS INJURIES.**

## SAVE THESE INSTRUCTIONS.

### INSTALLING BACK

1. Depress detent buttons on back piece and insert into commode to the desired position.
2. Ensure that the detent buttons are fully engaged and the back does not move.

### ADJUSTMENT INSTRUCTIONS

#### Height

1. Depress detent button on leg piece and move up or down to the desired position.
2. Ensure that the detent button is fully engaged and leg does not move.
3. Repeat steps 1 and 2 for the remaining three legs.
4. Make sure the commode sits level and all of the legs are at the same height.

#### Arms

To Drop Arm:

1. Press the push button located at the sides of the seat.
2. Flip arm down until it is completely out of the way.

To Raise Arm:

1. Pull up on arm until the button engages in the locked position.
2. Make sure arm is completely engaged prior to use of armrest.

## **OVER THE TOILET USE**

Use only as instructed by your healthcare professional.

1. Remove back by pushing detent buttons and pulling back away from commode.
2. Lift seat of commode and remove bucket.
3. Replace with splash guard.
4. Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.
5. Check to ensure that commode is level and stable before sitting.

## **CLEANING**

Your commode can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

### **Grip Cleaning Instructions**

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

**NOTE: Tips on this product contain dry natural rubber.**

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