GUARDIAN

PLATFORM ATTACHMENT

REF MDS86615P

User Instructions and Warranty

ACCESSOIRES DE PLATE-FORME

REF MDS86615P

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REF MDS86615P

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Consult your medical advisor for the proper use of this accessory. It is designed to be used on specific Guardian walkers that have two (2) attachment points that are 7/8" diameter. Use of this product on other than Guardian walkers may void warranty and may cause injury.

WARNING

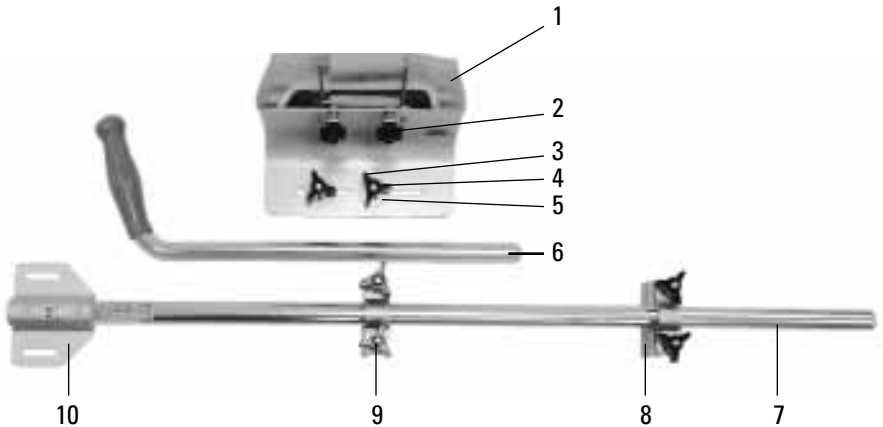
For safety, these precautions must be taken when using this product.

BEFORE EACH USE MAKE SURE:

1. You are correctly fitted.
2. Knobs are tightened and platform attachment is securely fitted to walker.
3. Grip, pad and hook and loop closure are in good condition. Replace immediately when worn.
4. Hand grip is attached securely and does not slip.
5. Nothing is loose or uneven and there are no rattles or cracks in the metal, plastic, etc. or any other irregularities.
6. Product is used only with the manufacturer's parts and accessories.
7. User does not have any handicaps, disabilities or is on medication, etc. that a doctor, therapist, and/or seller would deem to interfere with the safe usage of the product.

Failure to follow the above instructions may result in serious injury.

2. IMPORTANT PARTS

The platform attachment consists of the following parts:

- | | |
|---------------------------|---------------------------|
| 1. Armrest | 6. Handle |
| 2. Armrest knobs | 7. Post |
| 3. Angle adjustment knobs | 8. Metal plates |
| 4. Bolts | 9. Post mounting knobs |
| 5. Washers | 10. Post mounting bracket |

ATTACHMENT TO WALKER

1. Remove angle adjustment knobs and washers from armrest. Assemble by inserting bolts through post mounting bracket. Add washers and tighten knobs (Figure 1).
2. Insert handle for right or left hand use so that the armrest is on the inside of the walker (Figure 2). Tighten knobs slightly.
3. With the post to the outside of the walker, center metal plates on top of upper and lower spreader bars (Figures 3 & 4). Tighten post mounting knobs so that platform attachment is securely in place.
4. Adjust handle to desired length and tighten armrest knobs securely.
5. Angle of armrest can be adjusted using the angle adjustment knobs (Figure 5). Consult your medical professional for proper adjustment.
6. Make sure all knobs are tightened and platform attachment is securely fitted to walker.

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

GRIP CLEANING INSTRUCTIONS

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

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Figura 5

LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline Industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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