LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline Industries, Inc. Guardian Signature is a trademark of Medline Industries, Inc.

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GUARDIAN[®]
• S I G N A T U R E [™] •

**HEMI-WALKER
USER INSTRUCTIONS
& WARRANTY**

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REF G07771

WARNING

For safety, these precautions must be taken when using this product.

Before each use make sure:

- Sidestepper is adjusted to correct height.
- Detent buttons are fully engaged.
- Tips are in good condition. Replace tips immediately when worn.
- User weight does not exceed 250 lbs.

Use only as instructed by your healthcare provider.

WARNING

Always use extreme care when using this product. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

FAILURE TO FOLLOW THE ABOVE INSTRUCTIONS MAY RESULT IN SERIOUS INJURY.

SAVE THESE INSTRUCTIONS.

USER INSTRUCTIONS

Use only as instructed by your healthcare professional.

Opening and Closing

1. Open and close by moving handle located on lower half of Sidestepper.
2. In the open position, the handle should be horizontal and rest on the lower cross brace.
3. In the closed position, the handle should be vertical and the two halves of the frame will be folded against each other.
4. Be sure that the handle is in contact with the lower cross brace before using.

Height Adjustment

1. Adjust the Sidestepper so that when the arm is extended, the handle is at wrist height.
2. Depress spring button and slide leg up or down.
3. Engage spring button into hole that provides correct height.
4. Repeat for each leg, making sure Sidestepper sits level.

CLEANING

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

NOTE - Tips On This Product Contain Dry Natural Rubber.

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